

In light of the current situation with COVID-19, with regard to cleaning your Raizer or Raizer II, we would recommend using hygiene sleeves (part no. 103741). These sleeves can go over the seat of the Raizer or Raizer II and can simply be thrown away once used. This is primarily for the for seat, however you could also attach theses to the backrest should you wish. In both instances, these would need to be placed on the device prior to lifting the patient. Below is a link showing the hygiene covers being applied.

[Hygiene Cover for Raizer](#)



After each use we would also recommend thoroughly cleaning each part of the device as per the manufacturer guidelines below:

12.1 Cleaning



IMPORTANT! General cleaning may only be done using a well-wrung cloth or sponge.

The Raizer is **NOT** waterproof and therefore must not be immersed in water or flushed with running water. It is however acceptable to clean the Raizer with a damp cloth.



The Raizer must **NOT** be flushed with water



The Raizer must **NOT** be immersed in water

You can disinfect your Raizer by using the below disinfectants recommended:

Hydrogen Peroxide Aq.	1,0% (max.) (diluted hydrogen peroxide)
Ethanol based products	70-85 v/v% alcohol
Soap solutions	PH value max. 9

NEVER use a high-pressure cleaner or other power flushing or running water, as this may cause permanent damage to the lifting chair.

